## Portable Oxygen Concentrator User Manual

Certificate of Quality
Inspected By:
Passed

\*All languages are subject to English, and the manufacturer reserves the right of Final interpretation.

Please read the instructions carefully before use.

## Safety Information

In case of a possible power failure or equipment failure, users who are in urgent need of oxygen must be equipped with other standby oxygen supply devices(such as oxygen cylinders, oxygen bags,etc.).Note that this product cannot be used for life support or life extension! Not suitable for Newborns or infants!

The product is only used for oxygen supplement at high altitude or in low oxygen environments and cannot replace medical equipment.

Oxygen supports combustion.Thus smoking is prohibited when the equipment is working!

Do not use the equipment in high temperatures, or when there are sparks or open flames in the environment!

To prevent accidents, do not let children operate the equipment alone. When breathing oxygen, do not wrap the hose around the neck; The elderly with mobility difficulties must be accompanied by a guardian during use.

This product separates and purifies oxygen directly from the air for breathing. Therefore, please do not use it in the environment containing oil smoke, toxic and harmful gases.

This product cannot be connected with other equipment not allowed by the manufacturer.

If any accessories are damaged or lost, please contact the equipment supplier.

It is prohibited to use lithium batteries that are not equipped with the product.

When the battery is not in use, please protect the interfaces on the battery and on the device .Do not contact them with the conductor or touch them directly with hands.

Remove the battery before the equipment is discarded and contact the relevant departments to properly dispose of the equipment and accessories according to the relevant regulations of the local government.

## Definitions of the symbols

Explanation of identification symbols such as graphics, symbols, and abbreviations related to the use process of this product.

Symbols	Description	Symbols	Description
Warning	Important safety information for hazards that might cause serious injury	Caution	Important information for preventing damage to the device
	AC power		On/Off
	No smoking		Increase flow
	Keep away from flame		Decrease flow
	Refer to files delivered with the unit	8	8 Current setting. Settings1~8
	No oil or grease		Warning
	Do not disassemble		Symbol for the marking of electrical and electronics devices according to Directive 2012/19/EU.
	This way up		Type BF Applied Part
	Stacking limit by number		Class II
	Fragile		Observe the instructions for use
	Temperature limit		No flip
	Electromagnetic interference may occur	IP21	IP21 Protects against solid objects over 12.5mm and direct sprays of water up to 15°of vertical.

## Content

Introduction .....	1
Intended Use and Contraindications for Use .....	1
Main Structure .....	2
Display and Alarm .....	3
Buttons .....	5
Installation and Use .....	5
Operating Instructions .....	6
Clean,Care and Maintenance .....	7
Specifications .....	11
Electromagnetic Compatibility .....	13
Troubleshooting .....	13
Packing List .....	14
Unboxing Instructions .....	14
Software information .....	14
Warranty Statement .....	14

## Introduction

### 1. Working Principle

This product is powered by DC 20V power supply or 14.4V battery. Equipped with a pure copper oil-free compressor and a high-quality French molecular sieve, it uses air as raw material to produce high-purity oxygen through a new Pressure Swing Adsorption method(PSA) at room temperature. This is a totally physical method which is stable, safe and reliable.

### 2. Product Applications

Only for health care,and cosmetic oxygen therapy, not suitable for medical use.

### 3. Product Structure

Full plastic shell, safe and reliable;

Comes with a removable battery pack, easy to install and replace;

Small in volume and light in weight, the product can be easily carried around;

The product has a function of accumulating use time, so you can keep track of how long the machine has been used;

Low oxygen concentration alarm function lets the user know oxygen production status in real time.

## Intended Use and Contraindications for Use

### Intended use:

The device separates oxygen from air via PSA technology and delivers highly purified oxygen(> 90%(V/V) for users requiring oxygen therapy or supplemental oxygen.

⚠ **Warning:** Follow the guidance of a professional doctor when using.



⚠ **Warning:** Do not use this device in the presence of flammable anesthetics.

### Contraindications:

This device is not intended for users suffering oxygen intoxication and/or oxygen susceptibility.

⚠ **Warning:** Follow the guidance of a professional doctor when using.

## Main Structure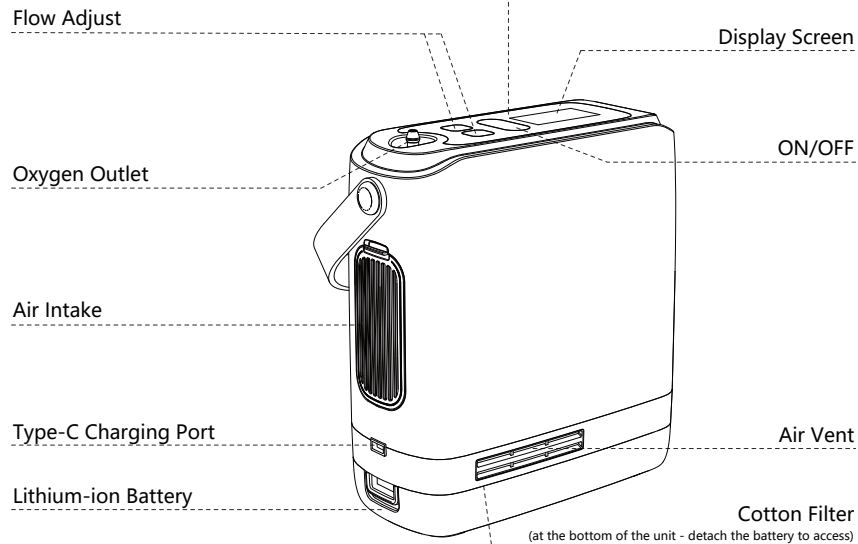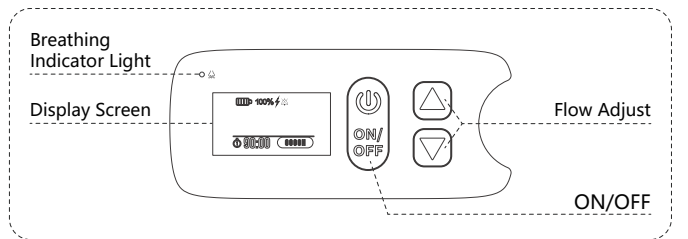


△ It is normal to have warm air coming out of the exhaust.

△ **NOTE:** Do not block air intake or air exhaust vents.


△ **NOTE:** All pictures are for illustration purposes only.

## Display and Alarm

### User interface: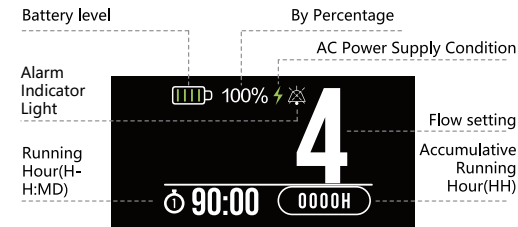

### Alarm:

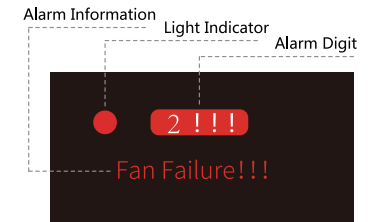△ **NOTE:** Display will vary with status of alarm, sound, function, flow setting.

### Alarm indicators:

Audible alarm: SPL ranges are 45-80dB at 1m.

Portable Oxygen Concentrator has faults display function. The alarm includes:

#### High priority:

##### (1) Overheat:

The alarm indicator flashes (flashing frequency is 2Hz), the device emits an alarm sound, the red indicator lights up, "1!!!" and "High Temperature!" are displayed on the screen.

##### (2) Fan failure:

The alarm indicator flashes (flashing frequency is 2Hz), the device emits an alarm sound, and the red indicator lights up, "2!!!" and "Fan Error!!!" are displayed on the screen.

##### (3) Compressor failure:

The alarm indicator flashes (flashing frequency is 2Hz), the device emits an alarm sound, and the red indicator lights up, "3!!!" and "Compressor Failure!!!" are displayed on the screen.

#### Medium priority:

##### (1) Low oxygen concentration ( $\leq 82\%$ ):

The alarm indicator flashes (flashing frequency is 0.5Hz), the device emits an alarm sound, and the yellow indicator lights up, "4!!!" and "Low Oxygen Concentration!!!" are displayed on the screen.

##### (2) Low pressure:

The alarm indicator flashes (flashing frequency is 0.5Hz), the device emits sound, and the yellow indicator lights up, "5!!!" and "Low Pressure!!!" are displayed on the screen.

##### (3) Low Battery:

The alarm indicator flashes (flashing frequency is 0.5Hz), the device emits an alarm sound, and the yellow indicator lights up, "6!!!" and "Low Battery!!!" are displayed on the screen when the battery level is  $< 5\%$  and the device is not connected with external power supply.

##### (4) No Breathing:

When breathing pulses are not detected after 4 minutes of startup, the alarm indicator flashes (flashing frequency is 0.5Hz), the device emits an alarm sound, and the yellow indicator flashes, "7!!!" and "No Breathing!!!" are displayed on the screen. During operation, if breathing pulses are not detected within 60 seconds, this alarm will also be triggered.

## Low priority

### (1) Warm up:

When the oxygen concentration may not have reached 90% in 120s upon startup, the device emits an alarm sound, and the yellow indicator light is constantly on, "8!" and "Warm up in progress!" are displayed on the screen.

### (2) Power interruption:

When the adapter is connected/removed, a "beep" will sound and power supply condition will display on the screen. The alarm shall restore automatically in the case power supply is disconnected less than 30s.

The visual alarm should be accurately and clearly detected at a distance of 1m from the device or operator. The audible alarm shall verify the presence of pulse frequency at a distance of 1 meter or where operator is expected, and at least 3 additional sets of harmonic audible alarm signals shall be detected at a range of 150Hz to 400Hz. Alarm level: there are three categories based on severity: high priority, medium priority, and low priority. High priority SPL $\geq$ medium priority SPL $\geq$ low priority SPL $\geq$ 45dB(A). SPL range at 1m is 45-80dB(A). Alarms are technical.

### Alarm and reminders:

No	Description	Audible	Status	Priority	Solutions
1	Overheat	Continuous "beep-beep-beep"	1. Audible alert 2. Flashing red light	High	① Place the device away from heat source or somewhere cool. ② Make sure it's well ventilated. ③ Contact equipment provider.
2	Fan failure	Continuous "beep-beep-beep"	1. Audible alert 2. Flashing red light	High	Contact equipment provider
3	Compressor failure	Continuous "beep-beep-beep"	1. Audible alert 2. Flashing red light	High	Contact equipment provider
4	Low oxygen level( $\leq$ 82%)	"beep-beep-beep"	1. Audible alert 2. Flashing yellow light	Medium	Contact equipment provider
5	Low pressure	"beep-beep-beep"	1. Audible alert 2. Flashing yellow light	Medium	Contact equipment provider
6	Battery level $\leq$ 5% and no external power is connected	"beep-beep-beep"	1. Audible alert 2. Flashing yellow light	Medium	Connect the AC power to charge the battery.
7	No breath is detected in 4mins upon startup	"beep-beep-beep"	1. Audible alert 2. Flashing yellow light	Medium	Use the unit with a cannula.
8	O <sub>2</sub> concentration is not reaching 90% in 120s upon startup	"beep-beep"	1. Audible alert 2. Yellow light is on	Low	Operate after warm-up is completed
9	Power outage	"beep"	1 Audible alert	Low	NA

## Buttons

ON/OFF:

Press and hold the button for 3 seconds to turn on the device. Press and hold the button for 3 seconds to turn off the device.

Short press the ON/OFF button to silence the alarm.

Flow setting:

Press "△" or "▽" to adjust to the desired flow rate. The digit shown on the display is an indicator for the current flow setting. There are 8 settings, from 1 to 8.

△ **Note:** When the device is initially turned on, the default flow setting is 3.

△ **Note:** Check carefully the display and the indicators, once an alarm is heard.

## Installation and Use

When using for the first time, please remove all packaging materials and keep plastic bags and other items out of reach of children to avoid the risk of suffocation.

Carefully read the instruction manual before use, ensure that accessories are installed in Place and operated and used correctly.

Please connect the battery when used. The battery is protected, and the use of a battery connection for a long time will not damage the battery.

To avoid danger of electric shock, do not touch with wet hands any live parts such as plug power cord, and adaptor.

Use of damaged power cord or plug can cause fire or electric shock.

It is not allowed to stack heavy items on this device, otherwise the device can be damaged.

Avoid contact with the exhaust vent or any heat source for a long time to avoid risk of burn.

Please set the appropriate flow gear according to the physician's suggestion.

This device should be positioned in a well-ventilated place so that the air intake or the air outlet vents are not blocked, otherwise the device performance can be impaired or fire can occur.

Do not leave the device exposed to direct sunlight which can lead to excessive heat.

Keep this device away from fire or smoking to avoid danger of fire.

Operation in extreme humidity conditions can lead to as shorter service life of the molecular sieve.


Keep this device away from pollutants, smoke, and substances that are flammable, explosive or volatile, such as ethanol and petroleum to avoid risk of fire or explosion.

Always operate the device in an upright position.

The product must be allowed to stand for at least 4 hours at an ambient temperature of 20°C after being removed from its minimum or maximum storage temperatures to ensure proper functionality.

Improper operation may cause damage to your device and void the warranty.

Removable parts connection: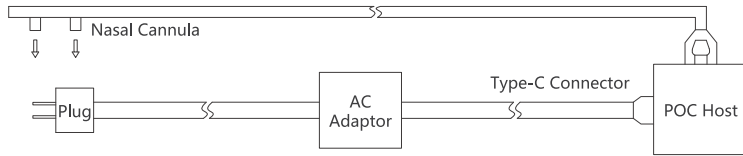

## Operating Instructions

### 1. Power supply

Select the power supply as the operating environment allows.

**A. Using battery: Install the dedicated battery for this device into the device battery interface.**

- ① Align the battery with the left end of the front of the device.
- ② Slide in from left to right, when a clicking sound is heard, it indicates that the battery has been installed in place.

**B. Using AC power supply:**

- ① Connect the AC power core to the power adaptor and connect the input plug to power supply.
- ② AC power should be 100-240V ~ 50/60Hz.
- ③ Connect the outlet of the power adaptor to the power port receptacle.
- ④ When the device is powered, it will enter standby mode. (Never remove the battery when the device is operating on.) Adaptor supplied with this device conforms to IEC60601-1.

⚠ **NOTE:** When not using the battery, please pay attention to protect the battery interface on the battery and the battery interface on the device. Do not touch the conductor or touch it directly with your hands.

⚠ **WARNING:** Do not use any adaptor that is not supplied with this device or conforming to IEC60601-1.

⚠ **WARNING:** Do not use any lithium battery other than those supplied with this device.

### 2. Preheating

Press On/Off button for 3~5s and a warm-up period of approximately 15 minutes will initiate. After preheating, the device can be used normally.

⚠ **NOTE:** Do not connect cannula during device warm-up as the oxygen may not meet specifications.

### 3. Connect the nasal cannula

Connect the nasal cannula to the metal outlet on the concentrator and ensure it is fully inserted and secure. Make sure the cannula is not kinked or obstructed.

Users can contact the device seller for your nasal cannula or select one that fits the device and is registered under instruction from a physician. Otherwise, do not use it and contact the device seller.

⚠ **NOTE:** This device delivers oxygen in pulse mode. A nasal cannula must be connected when the device is in use.

⚠ **NOTE:** Make sure the nasal cannula is correctly connected and is not kinked or obstructed. This ensures that the oxygen concentrator can properly detect inspiration for oxygen delivery.

⚠ **NOTE:** Please follow the manufacturer's instructions for using the nasal cannula. Replace the nasal cannula according to the recommendations of the manufacturer or device seller, and other accessories can be purchased from the device seller.

⚠ **NOTE:** Do not use nasal cannula that are not compatible with this device or purchased without a device seller or under the guidance of professional medical personnel, in order to avoid affecting the normal operation of the device and the normal use of the user. Non-legitimate manufacturer's nasal cannula may cause adverse effects such as air leakage, poor gas delivery, and odor.

### 4. Flow setting

Select the flow setting as prescribed by your physician. Communicate to your physician the Performance of this device before a prescription is made up.

### 5. Shutdown

Press the on/off button for 2~5 seconds when the device is operating on and it will shutdown.

### 6. Explanation of other situations

Battery level and charging icon will display when the adaptor is plugged in/out.

Alarm indicators will illuminate and an alert will sound once the device comes across an issue.

⚠ **NOTE:** Pay attention to the device condition and alarms if any.

⚠ **NOTE:** Do not attempt to remove the battery while the device is still on.

## Clean, Care and Maintenance

### 1. Consumable maintenance cycle


Item	Recommended Interval	How to care	Remark
Cotton filter	Every 2 months	Replaced by user	Contact your device seller
Sieve bed column	Once a year	Paid replacement	Contact your device seller
Battery	Charge at least once every 2 months if not in use	Charged to 30%	Follow care instructions

⚠ **NOTE:** The intervals recommended above are just for reference. It may vary based on the operating environment and conditions. Pay attention to the device condition and alarms if any.


⚠ **Declaration:** Do not use lubricants other than those recommended by the manufacturer.

## 2. Cotton filter replacement


Turn off the device, disconnect the power source and remove the battery.  
Removing the cotton filter from the device and replace them with new ones.  
Be careful while removing the filter to avoid inhaling dust.  
Install the new intake filter component in the correct direction.  
Insert the battery  
Dispose of the used cotton filter properly.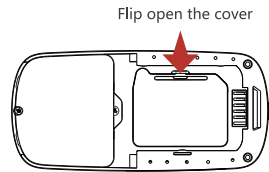


1) Remove the battery and open the intake filter component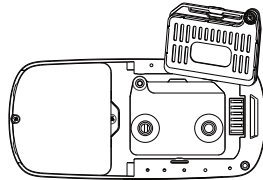

2) Remove the cotton filter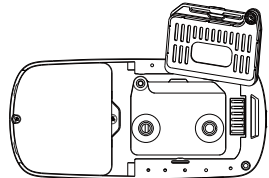

3) Install a new cotton filter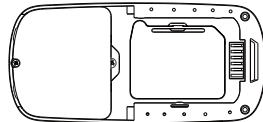

4) Install the battery

⚠ **NOTE:** Operate the device without filters or without replacing them for a long period of time may impair performance or damage the device. Replace the filters at intervals based on operating environment and conditions.


⚠ **NOTE:** The filters are consumable. Contact your device seller to order for replacement

## Molecular sieve unit replacement and maintenance


Please contact the seller when the device prompts low oxygen concentration and needs to Replace the molecular sieve unit.

Storage for a long period can lead to a shorter service life of the molecular sieve unit  
Operation in extreme humidity conditions can lead to a shorter service life of the molecular Sieve unit.


Use of molecular sieve unit near the end of its service life may adversely affect performance with increased pressure, noise or low concentration. Pay attention to alarms related to pressure and oxygen concentration.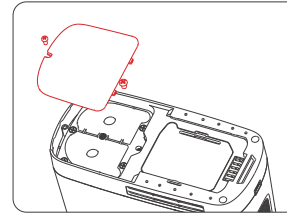


1) Open the cover plate of the molecular sieve unit and unscrew the screws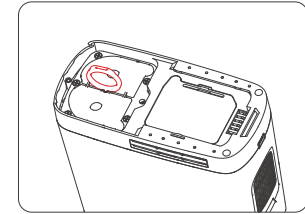

2) Screw in the put ring and put the molecular sieve unit upwards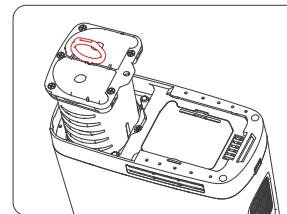

3) Replace the molecular sieve unit by a new one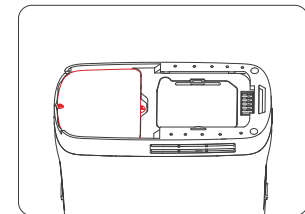

4) Screw the cover and reinstall the battery

⚠ **NOTE:** The molecular sieves unit are consumable. Contact your device seller to order for replacement.

⚠ Do not remove the sieve unit assembly when not necessary, otherwise it will lead to a shorter service life of the molecular sieve.

## 4. Battery pack care and replacement

This device is to be used only with the battery provided with it. DC: 14.4V, 6700mAh. This device includes one battery pack. User can contact the device seller to order as necessary.

Service life: 500 full charge/discharge cycles

Remove the battery pack from the device when the device is left unused for a long period. And keep the electrodes away from any conductor to avoid hazards of fire, etc. Keep batteries out of the reach of children.

For storage of batteries for a long period, a partial charged state of charge around 30% is recommended.

⚠ **WARNING:** Never use any lithium batteries other than those specified or provided by the manufacturer.

⚠ **NOTE:** The batteries are consumable. Contact your device seller to order for replacement.

## 5. Nasal cannula replacement

Additional cannula can be purchased. Make sure the cannula:

- is specified or recommended by the device seller or licensed clinician.
- is legally registered.
- reaches a flow rate of 5 L/Min.
- fits the oxygen outlet of this device well.
- Please wear correctly and use the nasal cannula according to cannula instruction.

## 6. Cleaning

Turn off the device, disconnect from all power supplies and remove the battery pack before any cleaning. Always disconnect AC power supply from the wall before disconnecting the AC power supply from the oxygen concentrator.

Care should be taken to prevent the oxygen concentrator from getting wet or allowing fluids to enter the device. This can cause a malfunction or shutdown, and cause an increased risk for electric shock.

Do not use liquid directly on the device. A list of undesired chemical agents includes but is not limited to alcohol and alcohol-based products, concentrated chlorine-based products (ethylene chloride), concentrated oxidant-based products.


The air intake filter is recommended to be cleaned weekly to ensure the ease of air flow. Use a damp cloth or sponge with a mild household cleaner on the exterior cover, control panel and power cord, and wipe them dry.

Do not allow any liquid to get inside the concentrator. Pay special attention to the oxygen outlet to make sure it is free from dust water, and particles. Do not apply any organic solvents that are flammable or explosive for cleaning.

Make sure the device is dry thoroughly before use.

⚠ **NOTE:** Remove the battery before disposal. Dispose of the device and accessories according to local regulations.

⚠ **NOTE:** The Picture below is for illustration purpose only.

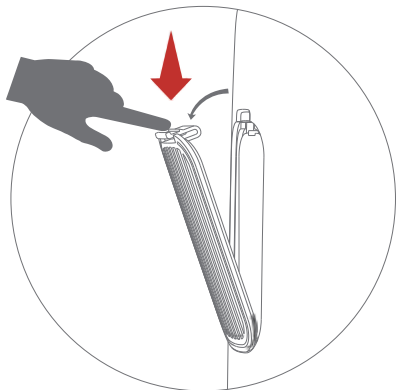## Specifications

Product	Portable oxygen concentrator							
Model	NT-08G							
Flow settings	Setting 1-8							
O <sup>2</sup> delivered by	Nasal cannula							
Dimensions L×W×H	183×86×199 ( mm ) Tolerance : ± 10mm							
Weight	1.98kg±15% ( including the battery )							
Sound level	40-60dB ( A ) ( 1-8 setting ) Ambient noise: Around 35dB ( A )							
Warm up	15 minutes to reach desired flow and concentration at setting 8 and in 25°C							
O <sup>2</sup> concentration	93±3% ( at all settings )							
Home use power supply	AC100-240V , 50Hz/60Hz , DC output : 20V							
Car use power supply	DC Input : 11-16V , DC output : 20V ; Rate Power : 120W							
Battery	14.4V , 96.48Wh , 6700mAh							
Battery duration	Setting1	Setting2	Setting3	Setting4	Setting5	Setting6	Setting7	Setting8
	≈4H ( ±30% )	≈2.7H ( ±30% )	≈2.2H ( ±30% )	≈1.6H ( ±30% )	≈1.5H ( ±30% )	≈1.4H ( ±30% )	≈1.3H ( ±30% )	≈1.2H ( ±30% )
Outlet pressure	≤200kPa							
Oxygen delivery	Pulse (not continuous)							
Battery recharging time	≤2.5H							

NT-08G Pulse Volumes Per Flow Setting ( mL/breath±15% )								
Breaths Per Minute	1	2	3	4	5	6	7	8
10	20	40	60	80	100	110	120	130
15	13.3	26.7	40	53.3	66.7	73.3	80	86.7
20	10	20	30	40	50	55	60	65
25	8	16	24	32	40	44	48	52
30	6.7	13.3	20	26.7	33.3	36.7	40	43.3
35	5.7	11.4	17.1	22.9	28.6	31.4	34.3	37.1
40	5	10	15	20	25	27.5	30	32.5
Total Volume Per Minute ( mL/min )	200	400	600	800	1000	1100	1200	1300

Operating Conditions	Temperature : 5°C~40°C Humidity : 15 ~ 93%R.H Atmosphere pressure : 86kPa ~ 106kPa
Transport & Storage Conditions	Temperature : -10°C ~ 40°C Humidity : 15 ~ 93%R.H Atmosphere pressure : 86kPa ~ 106kPa
Transportation	Keep dry. Handle with care. Keep upright. No flip
Type of protection against electric shock	Class II
Degree of protection against electric shock	Type BF
Degree of Safety of Application in the presence of flammable or anaesthetic gases	Not suitable for use in a presence of a flammable anesthetic mixture with air or with oxygen or nitrous oxide
Protection degree	IP21

## Electromagnetic Compatibility

This device is designed to conform to YY 9706.102-2021 Medical electrical equipment - Part 1-2: General requirements for basic safety and essential performance and article 202 magnetic related in YY 9706.269-2021 Medical electrical equipment-Part 2-69. Particular requirements for basic safety and essential performance of oxygen concentrator equipment.

The user should install and operate according to the electromagnetic information supplied with this device. Keep the device while in use away from portable or mobile RF communications equipment including mobile phones and microwave ovens to avoid strong electromagnetic interference, otherwise device performance can be impaired. Refer to Appendix 3 for Guidance and manufacture's declaration.

## Troubleshooting

Problem	Possible cause	Recommended Solution
Concentrator does not power on when On/Off power is pressed	Battery is not installed properly	Remove the battery and reinstall it
	Battery is depleted	Use external power supply
	AC power supply is not connected properly	Check and ensure external power connection is secure
No oxygen	Cannula is kinked or obstructed	Check cannula and its connection to oxygen outlet
	Device failure	Contact your device seller
Low oxygen concentration	Sieve unit may require maintenance	Contact your device seller
Overheat	Temperature is high. Device internal temperature is high	① Position the device away from any heat source and in a well-ventilated place. ② Ensure the air inlet and exhaust are not blocked. ③ Contact your device seller.
Fan error alarm	Fan error	Contact your device seller
Compressor failure alarm	Compressor failure	Contact your device seller
Oxygen percentage below 82%	Oxygen concentration is low	Contact your device seller
Low pressure	Air leakage	Contact your device seller
Low battery	Battery level: ≤5%	Connect the adapter to charge the battery
All other problems	Perform a full power cycle by removing the battery and disconnecting any AC/DC external power source. Then reinstall battery and power cycle is complete. Contact your device seller.	

### The following states are not faulty

It is normal that air emitted from the air outlet vent become warmer when the device is operating for a long period of time or in a warm environment. Device will alert when it becomes overheating because of blocked exhaust vent or any other reason.

## Packing List

No.	Item	Qty	Note
1	Oxygen concentrator host	1	Standard configuration
2	Battery	1	Standard configuration
3	AC adaptor	1	Standard configuration
4	Carry bag	1	Standard configuration
5	User manual (with quality certificate and warranty card)	1	Standard configuration
6	Ring pull	1	Standard configuration
7	Nasal cannula	1	Standard configuration

## Unboxing Instructions

- ① Contact freight company and device seller if damage to package is found.
- ② Verify that all of the components listed are included in the package. If any items are missing or defective, contact your device seller or after-sales service immediately.
- ③ Keep all packages for storage or transportation.

## Software information

This software is applied to portable oxygen concentrators, providing high concentration oxygen by controlling components such as compressors and solenoid valves, and sensing human respiration through sensors to deliver high concentration oxygen when the user inhales. At the same time, the control board continuously processes the collected data and displays it in real-time through display components. If a fault event occurs, the alarm system will sound an alarm. This software is an independent embedded software that can only run on the motherboard components of medical portable oxygen concentrators. It has no data transmission function and does not have external interfaces or access control interfaces. The software release version is 1.0.

## Warranty Statement

The Oxygen concentrator host (Excluding consumables such as molecular sieve): 1 year from the date of purchase.

Battery pack and AC adaptor (power cord): 1 year from the date of purchase.

Any service under this warranty shall be provided by an authorized service center.

Any repair under this warranty with exception of those caused by human factors including but not limited to misuse, abuse, improper use will be provided for free.

Any out of warranty repair will be provided for a fee. If damage is caused due to improper use, paid repair services shall be provided.

Consumable intake filter unit and molecular sieve unit are not covered by the warranty.

## Warranty Card

Dear user:

Please fill out the card and have it well kept. Warranty is valid with this card only.

Customer Name: \_\_\_\_\_

Contact: \_\_\_\_\_

Product: \_\_\_\_\_ Model: \_\_\_\_\_

Purchased on: \_\_\_\_\_

Purchased at: \_\_\_\_\_

SN: \_\_\_\_\_

## Maintenance Record

Date	Trouble

### Note:

1. The warranty card and receipt will be required for maintenance.
2. The Oxygen concentrator host (Excluding consumables such as molecular sieve): 1 year from the date of purchase.
3. Conditions listed as below will void the warranty and lead to paid service:
  - ① No receipt.
  - ② Modified receipt
  - ③ Accidental damage or improper use.
  - ④ Any repair other than authorized by the manufacturer.
4. Paid service is available for those expired.

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## VISION

Empowering Global Families for Healthy Living

## MISSION

Our brand strives to meet the home wellness and healthcare needs of global customers through competitive product solutions and services, maximizing customer value

<https://varoninc.com>

@VARON Inc.com

@varonoxygenconcentratorshop

Machines purchased from any platforms like Amazon and All Official Channels can get complete and professional technical support through Varon official after- support department.

### Varon US-Based Tech Support

TEL: +1 213-370-6080

E-mail: [support@varoninc.com](mailto:support@varoninc.com)

Attention: If you have any questions about the machine, please co via email and include your order number and a description of the

### WARRANTY DETAILS FOR EACH MACHINE

Model	Whole Unit Warranty Duration	Technical Supp
VP-1/NT-1 Pro	1 year	Lifetime
VP-2/NT-2 Pro	1 year	Lifetime
VL-1/NT-3	1 year	Lifetime
VL-2/NT-5	1 year	Lifetime
VT-1	1 year	Lifetime
VP-6/NT-06	1 year	Lifetime
VP-8G/NT-08G	1 year	Lifetime
VP-08 lite/YK-1006S	1 year	Lifetime
VH-1/101W	1 year	Lifetime
VH-2/105W	1 year	Lifetime
VH-2 Pro/Y-203W	1 year	Lifetime
VH-3/NT-04	1 year	Lifetime
VH-4	1 year	Lifetime
JAY-3AW	1 year	Lifetime
Serene 5/JAY-5AW	1 year	Lifetime